


## Ajusting the locking hook

Having trouble opening and closing the locking hook on your Eloflex? Do you want to have less, or perhaps more resistance when locking it? By tightening or releasing the two springs next to the lock hook you can easily adjust it. See the picture below.

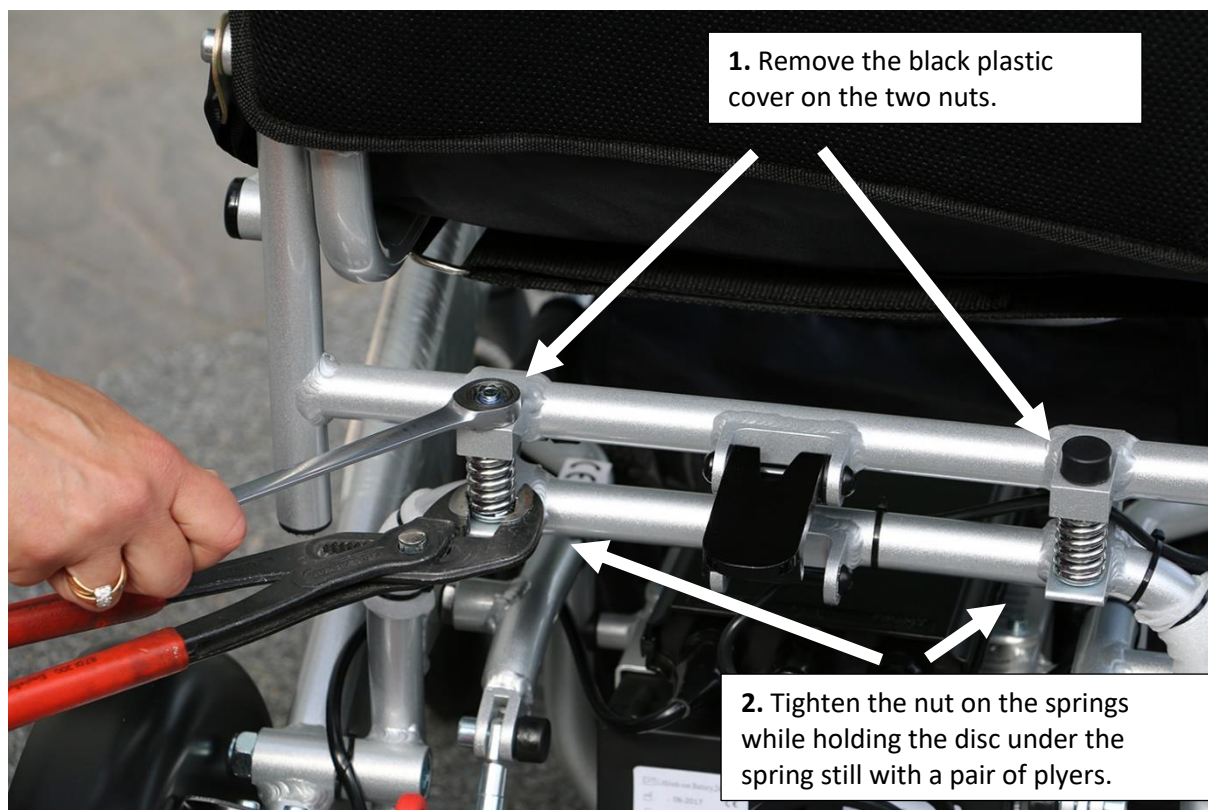

### Important:

It's important that there is enough resistance when pulling the lock up and down. There should be enough resistance so that it can point straight out on its own like the picture above. It's important so that the hook can stay in its locked position even if the springs are pressed down. Adjust the inertia of the locking hook simply by adjusting the screws on each side that hold it in place.

