## ELG.FLEX

## **Startup guide Eloflex**



1 Open the box, remove the foam on top. Lay the box down and roll out your Eloflex (see picture).



**2** Remove the box and protective plastic. Recycle it via plastic waste.



3 Unfold the Eloflex by pulling the backrest up and back. Lock the hook under the backrest (see the instruction manual).



Plug the loose cable into any available outlet on the battery. Tighten the locking ring firmly around the outlet.



**5** Cut the protective packaging material on both sides with scissors (see picture).




Remove the protective packaging material on the rear arch above the motors by cutting the zip-ties.

## ELC.FLEX



7 Take out the joystick from the bag underneath the seat.

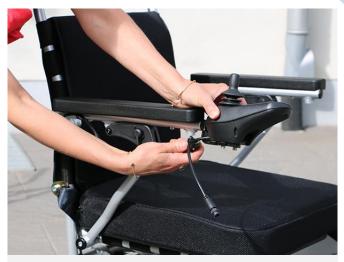

8 Mount the joystick on your preferred side.
Tighten the screw so the joystick doesn't move.



**9** Carefully assemble the contacts on the joystick cable after checking the guiding tracks. Do not bend the pins! Tighten the locking rings.



**10** Pump the rear wheels of the F and P to 2,5 kg air pressure



11 If the locking hook feels stiff and difficult to open/close, it can be adjusted by tightening the two springs (see picture).



12 The inertia of the footrest can easily be adjusted by tightening or releasing the two fixing screws. You can then lock the nut with loctite (see picture).